

# Status Mono

Immunoassay for the qualitative detection of infectious mononucleosis IgM heterophile antibodies in serum, plasma or whole blood




- FDA cleared with no age restrictions
- **CLIA WAIVED** for whole blood
- Single reagent procedure provides results in 8 minutes
- Fingertick whole blood convenience
- Superior 99.9% sensitivity for earlier IM detection, specificity of 98.8%
- Available in 10, 25 & 30 test kits; ideal for any practice size
- External QC negative and positive controls included (Item #68364 & 84M30)
- New plastic sample transfer pipettes for increased safety

#### CPT CODE


86308QW	Whole Blood
86308	Serum/Plasma

**STEP 1**  
Remove a test device from its pouch and place on a flat surface.

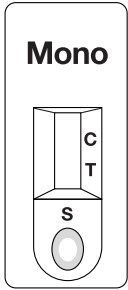


**STEP 2**  
Collect sample using the appropriate sample transfer pipette and add to Test Device.


- 25µL (red line) for whole blood
- 10µL (black line) for serum/plasma

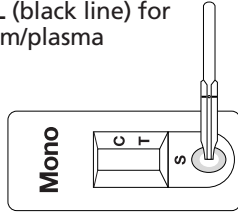



**STEP 3**  
Add 2-3 drops of Developer Solution into Sample Well.

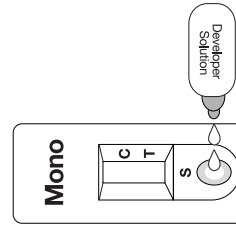

**STEP 4**  
Read the results at 8 minutes.

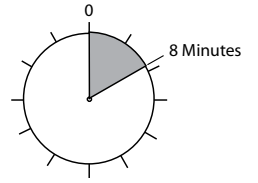


## INTERPRETATION OF RESULTS

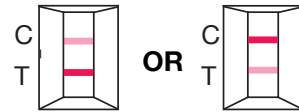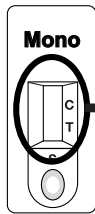


Two Lines  
Positive (+)

**Any visible T line  
is positive**

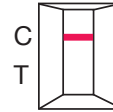

C Line only  
Negative (-)

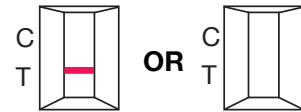

No C Line  
Invalid

**Repeat with new  
sample and device**

## ORDERING INFORMATION

PRODUCT	# OF TESTS	ITEM #
Status Mono (whole blood, serum or plasma)	25	68364
Status Mono (whole blood, serum or plasma)	30	84M30
Status Mono (whole blood only)	30	84W30
Status Mono (whole blood only)	10	84W10
CONTROL PRODUCT	# OF TESTS	ITEM #
StatusFirst Mono Controls	2 x 0.5 mL	200014



Infectious Disease • Drugs of Abuse • Cardiac Markers  
GI Health • Women's Health • Urology

Proudly made in the USA

For more information or to place an order, contact:

**lifeSign**

Authorized Distributor: [AmericanDrugTest.com](http://AmericanDrugTest.com)  
3482 Keith Bridge Rd, Cumming, GA 30041 | 800-915-7017