

Have these items ready before starting a test

Please refer to the package insert for detailed information regarding specimen collection and product specifications.



Test Device



Use the red line pipette for whole blood

Use the black line pipette for serum/plasma



Developer Solution



Timer

STEP 1

Open the foil pouch to remove the test device. .




Place the device on a flat surface



STEP 2


Collect a sample by fingerstick or venipuncture. Hold the pipette in a horizontal position to the specimen. **DO NOT SQUEEZE** the bulb of the pipette during collection. Allow the sample to flow up to the line on the pipette.

If serum or plasma specimen is used the pipette can be held horizontally to the serum or plasma collection tube.

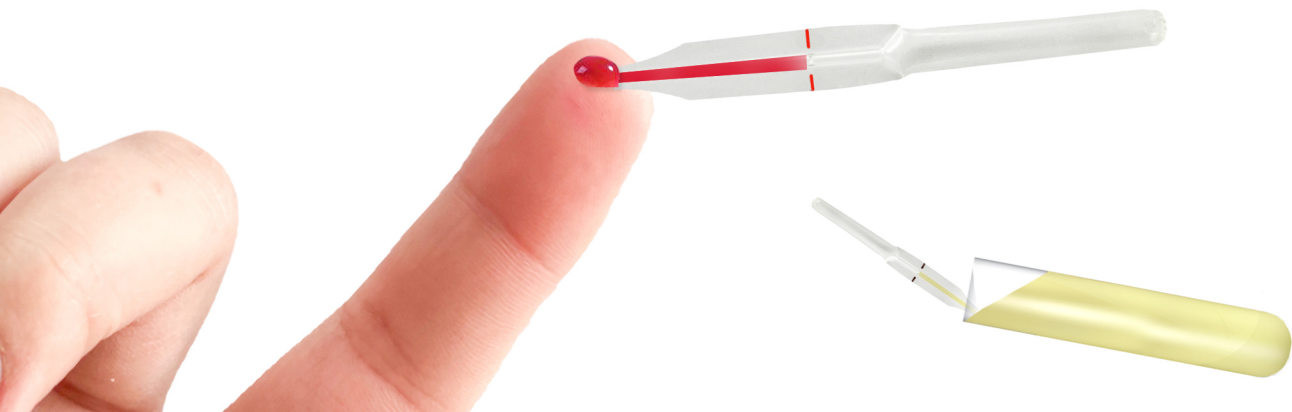

STEP 3

Once the pipette is filled to the line, hold the pipette in a vertical position and squeeze the pipette bulb to dispense the specimen into the upper area of the sample well.

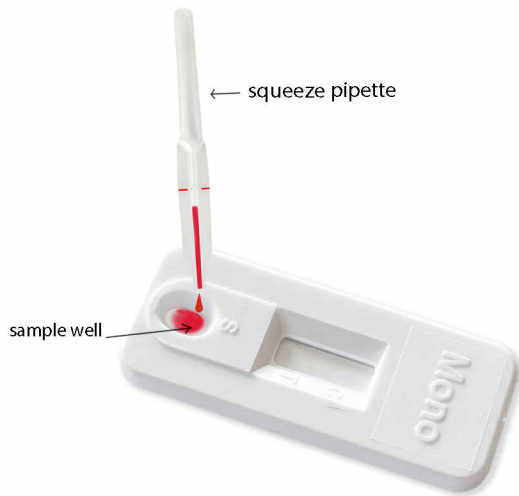

STEP 4

Add 2 drops of Developer Reagent into the lower area of the sample well

Set a timer for 8 minutes.



STEP 5

Read test results



**NEGATIVE
TEST**



**POSITIVE
TEST**



**INVALID
TEST**

No Control Line